Getting Started

First-time setup is done on the phone and takes about 15 minutes. We recommend having a sighted helper with you.

1 Take the phone out

Push the clip away from the phone and lift the phone out of the headset. Press and hold the power button marked by a yellow sticker until the phone vibrates. On Welcome, tap **Change Language** if you want another language, then tap **Next**.

2 Connect to Wi-Fi

Tap **Open Wi-Fi** on the phone. Choose the network and enter the password. Then press the **Back** button at the top left to go back to the onboarding screen. When Vista is connected, tap **Next**.

3 Turn on the remote

The remote is already paired. Press and hold the small power button between **Back (X)** and the stick for **1 second** until the blue light appears. Wait for "**Your remote is connected**" — this can take up to a minute. Then tap **Next**.

4 Put the phone back

When asked, put the phone back in the headset and close the cover.

5 Put on the headset

Put the headset on and adjust the straps: buckles toward the strap loosen, away from the headset tighten. Turn the **focus wheels** on top above each eye and the **IPD dial** until the picture looks clear and centered. Then follow the on-screen instructions. Iris will walk you through the rest of setup.

Getting Started videos and detailed instructions are online. Scan this QR code with your phone, or type the URL below.

irisvision.com/help-center/getting-started/

Need help? Call (855) 207-6665

Getting Started

English steps are on the other side. Other languages are below.

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